



V16 4/18

# Porcelain Conditioner

## FOR BONDING TO PORCELAIN CROWNS & JACKETS

---

**Intended Use:** Reliance Porcelain Conditioner is intended for use in the preparation of glazed dental porcelain for bonding.

Porcelain Conditioner is a silane coupling agent which chemically unites with the silicon in the porcelain and the acrylic in chemical, light cure and dual cure adhesives.

**Instructions for Use:**

1. Dispense one drop of Porcelain Conditioner to a mixing well.
2. Sandblast or roughen the porcelain surface with a fine diamond bur. Rinse and dry.
3. Using an applicator, apply one THIN coat of Reliance Porcelain Conditioner. Wait a minimum of 3-4 minutes. Lightly dry with air for 20 seconds. If warm air is utilized to dry, the wait time can be reduced to 1 minute.

**NOTE: A NEW APPLICATOR IS REQUIRED FOR EACH PATIENT**

4. Apply 1 coat of Assure Plus, lightly dry with air and light cure for 10 seconds. If Assure Plus is not available, the porcelain must be conditioned with Porc-Etch after step 1.
5. Place any light cure, dual cure or chemical cure paste on the bracket base and position on the tooth. If a light cure or a dual cure paste is used, position light and cure per manufacturers' instructions. If Rely-a-Bond® (no mix, chemical cure) is used, place a thin coat of Rely-a-Bond® Primer on the bracket base and the conditioned porcelain surface. Apply the Rely-a-Bond® Paste to the bracket base, position on the tooth, and press firmly. If Phase II® or Excel® are used, apply bracket and paste.



**PRECAUTIONS:** May cause skin sensation in susceptible persons. In case of contact with the skin, wash thoroughly with soap and water. Contact with other tissues: rinse area immediately with copious amounts of water for several minutes.

**CAUTION: DO NOT REUSE IF BOTTLE BECOMES CONTAMINATED. A NEW APPLICATOR IS REQUIRED FOR EACH PATIENT TO AVOID CONTAMINATION RISK.**



Porcelain Conditioner is flammable.



**Temperature Limitation:** Store at room temperature (15-30 Degrees C)

**To Reorder** Ref# PO - 8cc Porcelain Conditioner

**Warranty:** Reliance Orthodontic Products, Inc. recognizes its responsibility to replace products if proven to be defective. Reliance Orthodontic Products, Inc. does not accept liability for any damages or loss, direct or consequential, stemming from the use of or inability to use the products as described. Before using, it is the responsibility of the user to determine the suitability of the product for its intended use. The user assumes all risk and liability in connection therewith.

**Rx Only:** Federal law restricts this device to sale by or on the order of a dental professional.



*Reliance Orthodontic Products, Inc.*

1540 West Thorndale Ave., Itasca, IL 60143 U.S.A.  
1-800-323-4348 • [www.RelianceOrthodontics.com](http://www.RelianceOrthodontics.com)



EMERGO EUROPE  
Prinsessegracht 20  
2514 AP The Hague  
The Netherlands

Australian Sponsor: Emergo Australia  
Level 20, Tower II, Darling Park  
201 Sussex St.  
Sydney, NSW 2000 Australia

Scan for more information